




Breathe Easy Quick Start Guide

Insert Batteries

Insert Batteries with the positive end facing upward toward the mouthpiece

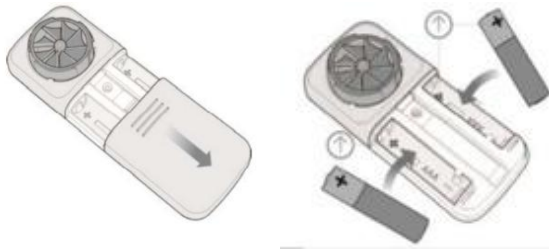

Download Breathe Easy


Search for ZEPHYRx Breathe Easy and install app for iOS, Google or Amazon



Pair your spirometer with the app



Agree to share your results

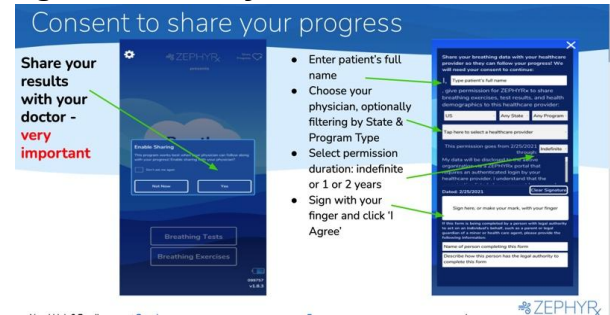

Enter your Health Demographics

All 5 categories must show a green check mark in order to submit

Sharing Enabled

Your Health Demographics

The following questions help us make a more accurate assessment of your breathing quality, based on your physical attributes.

Sex at Birth: ☐ Male ☒ Female

Birth Date: 1988 November 2

Weight: 170

Height: 5 8

Ethnicity: Caucasian

Submit

Take a Breathing Tests

Select 'FVC' to take a Breathing Test

Sharing Enabled

Breathing Tests

These PFTs (or pulmonary function tests) help you and your healthcare provider assess your lung function.

FVC
Forced Vital Capacity

View Past Results
Your Health Demographics
Export Your Results
View Trends
Sync Data

New here? Connect your ZEPHYRx and view the tutorial

Connect Help 09/07 v1.0.3